



ISO 13485:2016
ICMED 13485




ISO 9001:2015

WHO GMP
CERTIFIED


Diagnos HCV BI-DOT

Rapid Visual Test for the Detection of Antibodies
to Hepatitis C in Human Serum or Plasma

- ***Unique Combination of HCV Antigens
Core, NS3, NS4 & NS5***
- ***Detects all the subtypes of HCV***
- ***Results within 3 minutes***
- ***Based on Flowthrough Technology***
- ***No Instrument required***
- ***Built in Quality Control***
- ***Excellent Sensitivity 100 % & Specificity 100%***


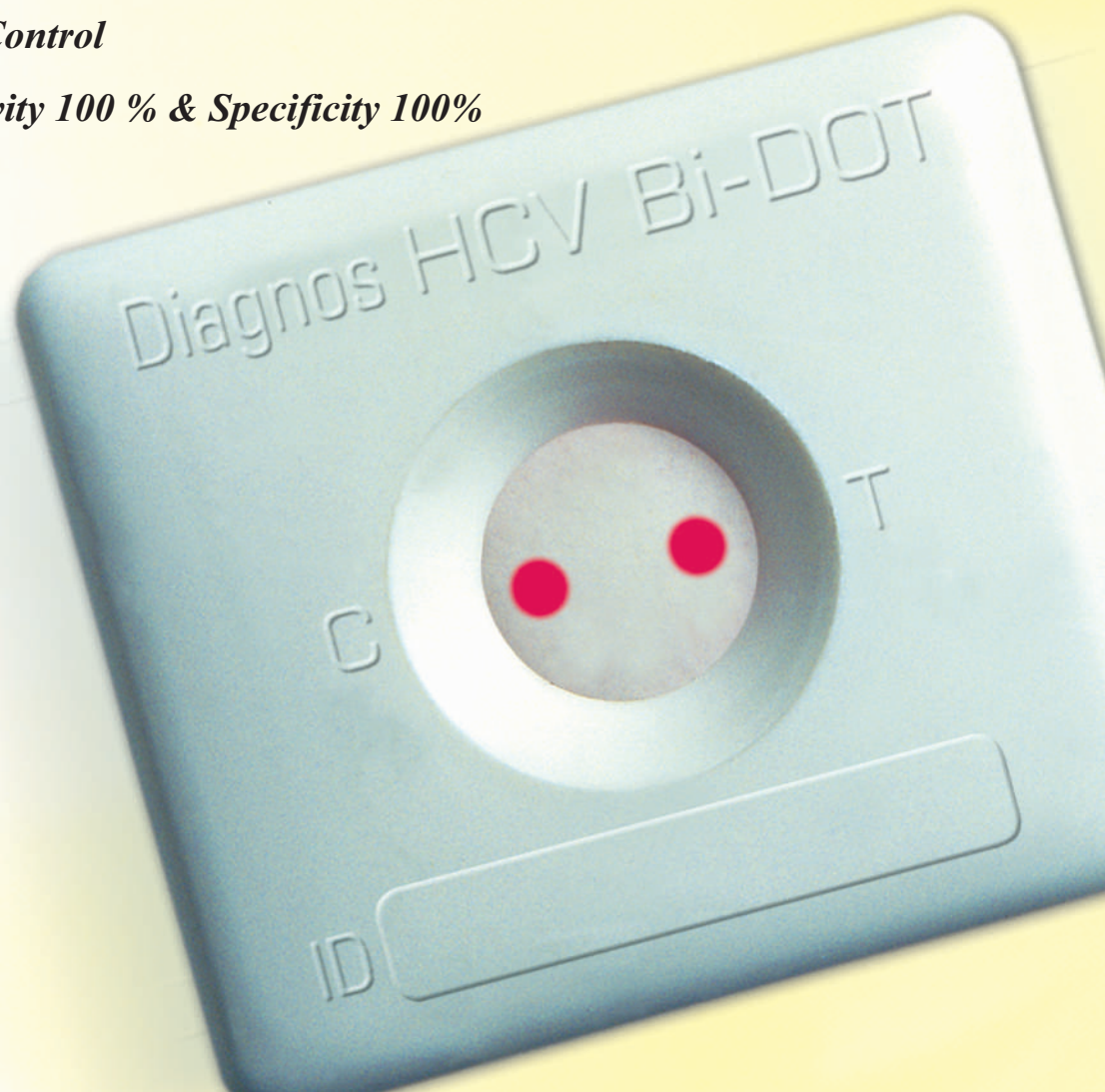


Principle

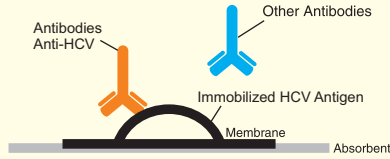


In flow through technology antibodies to HCV bind to the immobilized HCV antigen. Sample also contain other non-specific antibodies.

The non-specific antibodies are washed away by washing.

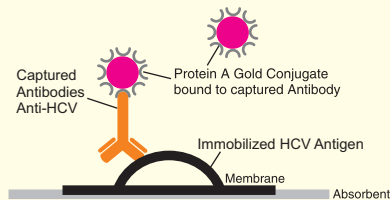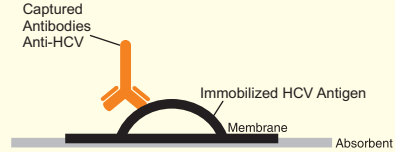

Protein A conjugate binds to the Fc portion of Anti-HCV antibodies


After adding the Protein A gold conjugate unbound conjugate is washed away during the washing step.

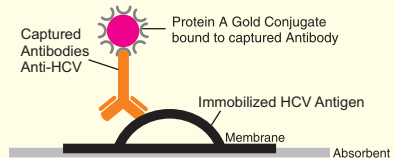


Test Procedure

- Add 3 drops of Buffer Solution.
- Add 1 drop of Patient's serum/plasma.
- Add 5 drops of buffer solution.
- Add 2 drops of conjugate solution.
- Add 5 drops of buffer solution.
- Read Results.


Results Interpretation

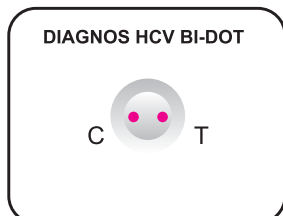


REACTIVE : for HCV antibodies

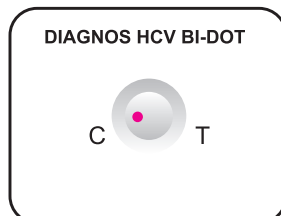

NON-REACTIVE

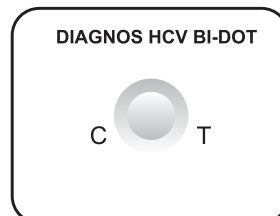

INVALID

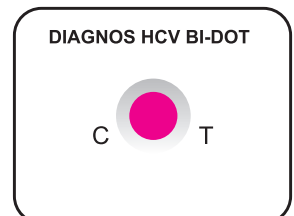

INVALID

Kit Presentation

10 Test Pack & 50 Test Pack

Also Connect with us on:



For further enquiries, Please contact:
J. Mitra & Co. Pvt. Ltd.

A 180-181, Okhla Indl. Area, Phase-1, New Delhi - 110 020, INDIA
Tel: +91-11-47130300, 500, 26818971-73

E-mail: jmitra@jmitra.co.in Website : www.jmitra.co.in