



ISO 13485:2016
ICMED 13485



ISO 9001:2015

WHO GMP
CERTIFIED

HEPACARD

One Step, Rapid, Visual & Qualitative Immunoassay for HBsAg detection



Sensitivity: 0.5ng/ml

In
Collaboration with
path U.S.A.
Program for Appropriate
Technology in Health

HEPACARD

Salient Feature


- One step, rapid, visual and qualitative immunoassay
- See-through device introduced for the first time in India
- No sample preparation required
- Ready to use
- Shelf Life 24 months at 2-30°C
- 100% user safe as there is no direct contact of user with sample & strip. Thus limits the infection risk.

Simple to Perform




Bring the HEPACARD foil pouch and serum samples to room temperature prior to testing.

Take out the required number of HEPACARD test devices from the foil pouch.

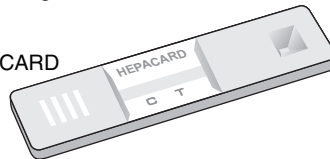


Label the test card with patient's name or identification number.

Add 2 drops (70µl) of serum/plasma sample into the sample well using the dropper provided.

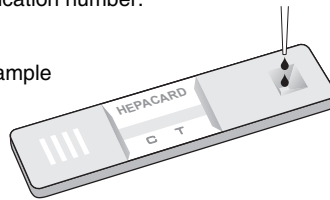


Allow reaction to occur during the next 20 minutes and **READ RESULTS**.

Test Interpretation

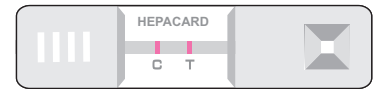

Reactive for HBsAg

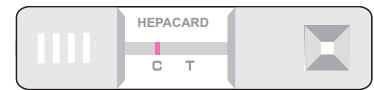

Non-Reactive for HBsAg



Invalid

Performance Characteristics

- It can detect all the **eleven** sub-types of HBsAg
- Sensitivity: 0.5 ng/ml
- **Evaluation Reports:**
 - **PATH (USA) Evaluation:** Sensitivity: 100%* Specificity: 100%*
 - **CMC, Vellore, India:** Sensitivity: 100%* Specificity: 100%*

*This information is provided for the Scientific Community Enquiring for an independent evaluation other than company's in house evaluation. It is not for commercial or promotional purpose.

Kit Presentation

20 Test Pack (Single Test Pouch packing)

100 Test Pack (Single Test Pouch)



For further information please contact:
J. Mitra & Co. Pvt. Ltd.

A 180-181, Okhla Industrial Area, Phase-1, New Delhi-110 020, INDIA
Ph.: +91-11-47130300, 47130500, 26818971-73
E-mail: jmitra@jmitra.co.in Internet: www.jmitra.co.in

Also Connect with us on:

