



ISO 13485:2016
ICMED 13485

HEPACARD

One Step, Rapid, Visual & Qualitative Immunoassay for HBsAg detection



HEPACARD
Antisera Anti-HBsAg dispersed on membrane

HBsAg TEST
HBsAg Testing in Human Serum/Plasma, No Reagents Required
(With Dessiccant)

Do not open the foil pouch until it attains room temperature
20°C - 30°C

**SENSITIVITY:
0.5 ng/ml**

J. Mitra & Co. Pvt. Ltd.
A-180-181, Okhla Industrial Area, Phase-1,
New Delhi - 110 020, INDIA

In
Collaboration with
path U.S.A.
Program for Appropriate
Technology in Health

Sensitivity: 0.5ng/ml

HEPACARD

Salient Feature


- One step, rapid, visual and qualitative immunoassay
- See-through device introduced for the first time in India
- No sample preparation required
- Ready to use
- Shelf Life 30 months at 2-30°C
- 100% user safe as there is no direct contact of user with sample & strip. Thus limits the infection risk.

Simple to Perform




Bring the HEPACARD foil pouch and serum samples to room temperature prior to testing.

Take out the required number of HEPACARD test devices from the foil pouch.

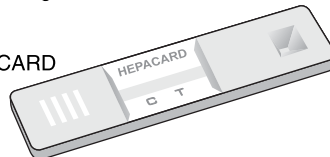


Label the test card with patient's name or identification number.

Add 2 drops (70µl) of serum/plasma sample into the sample well using the dropper provided.

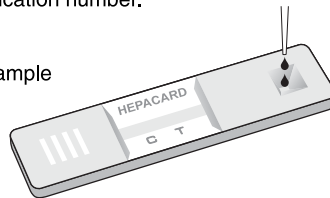


Allow reaction to occur during the next 20 minutes and **READ RESULTS**.

Test Interpretation

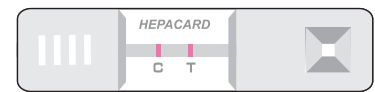

Reactive for HBsAg

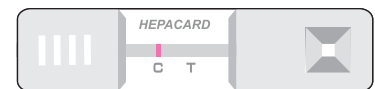

Non-Reactive for HBsAg



Invalid

Performance Characteristics

n It can detect all the **eleven** sub-types of HBsAg

n Sensitivity: 0.5 ng/ml

n **Evaluation Reports:**

● **PATH (USA) Evaluation:** Sensitivity: 100%* Specificity: 100%*

● **CMC, Vellore, India:** Sensitivity: 100%* Specificity: 100%*

**This information is provided for the Scientific Community Enquiring for an independent evaluation other than company's in house evaluation. It is not for commercial or promotional purpose.*

Kit Presentation

20 Test Pack

100 Test Pack



For further information please contact:

J. Mitra & Co. Pvt. Ltd.

A 180-181, Okhla Industrial Area, Phase-1, New Delhi-110 020, INDIA

Ph.: +91-11-47130300, 47130500

E-mail: jmitra@jmitra.co.in Internet: www.jmitra.co.in

Also Connect with us on:

