


HEPACARD

One Step, Rapid, Visual & Qualitative Immunoassay
for HBsAg detection

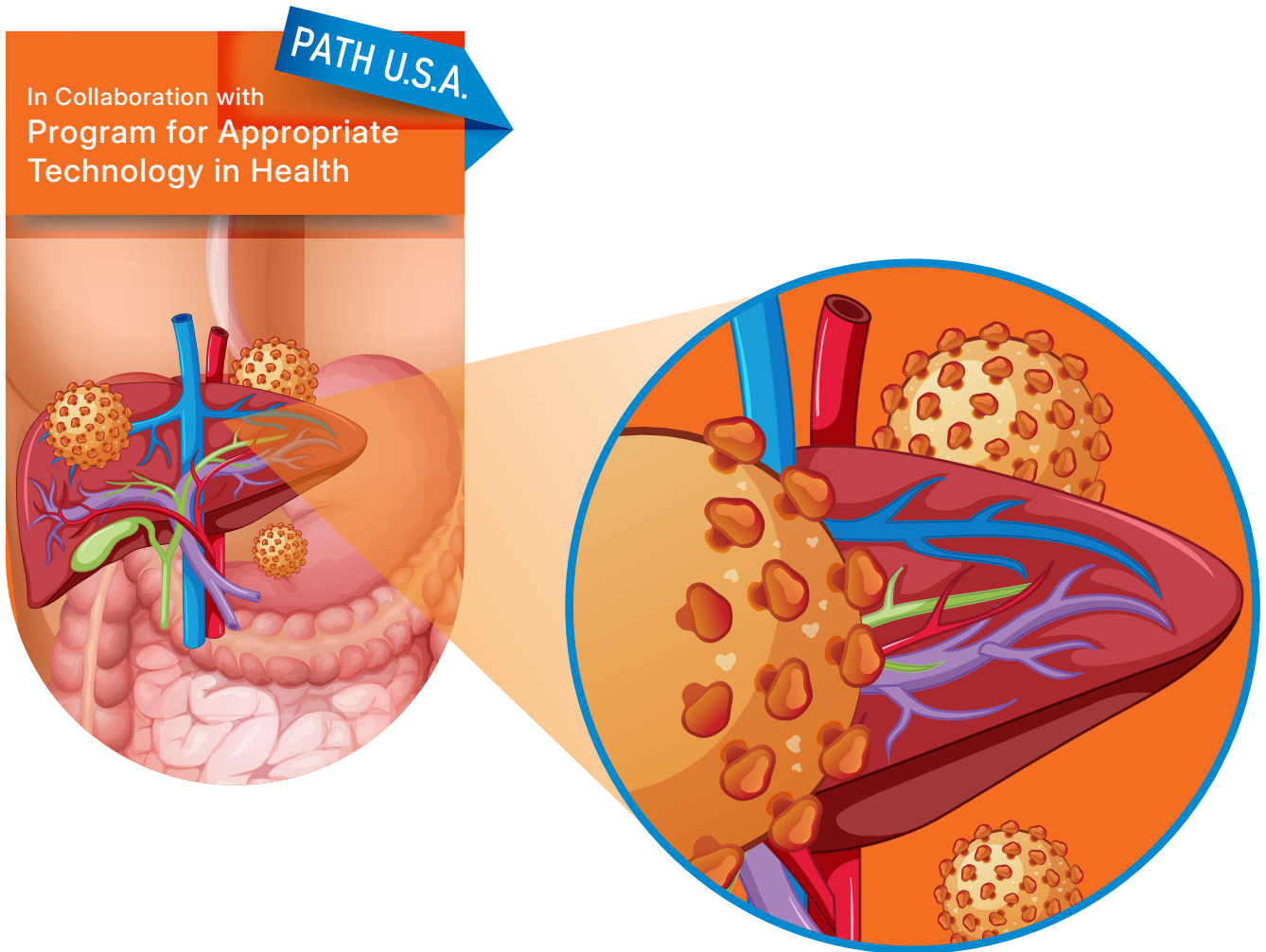

- One step, rapid, visual and qualitative immunoassay
- See-through device introduced for the first time in India
- No sample preparation required
- Ready to use
- Shelf Life 30 months at 2-30°C
- 100% user safe as there is no direct contact of user with sample & strip. Thus limits the infection risk.

Sensitivity: 0.5ng/ml

KIT PRESENTATION

20 Test Pack | 100 Test Pack



Scan QR Code
For more product &
service information




Since 1969

J. Mitra & Co. Pvt. Ltd.

.....a vision to serve mankind™

SIMPLE TO PERFORM

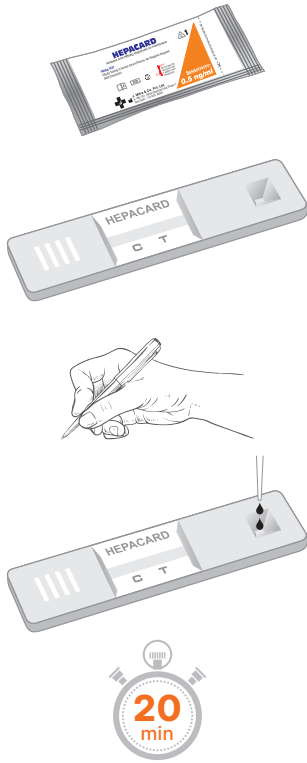

Bring the HEPACARD foil pouch and serum samples to room temperature prior to testing.


Take out the required number of HEPACARD test devices from the foil pouch.

Label the test card with patient's name or identification number.


Add 2 drops (70µl) of serum/plasma sample into the sample well using the dropper provided.

Allow reaction to occur during the next 20 min and **READ RESULTS.**

TEST INTERPRETATION

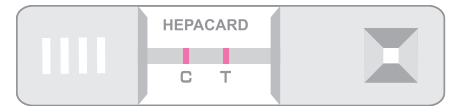

Reactive for HBsAg

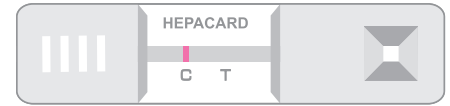

Non-Reactive for HBsAg



Invalid

PERFORMANCE CHARACTERISTICS

- It can detect all the eleven sub-types of HBsAg
- Analytical Sensitivity: 0.5 ng/ml

EVALUATION REPORTS

PATH (USA) Evaluation: Sensitivity: 100%* | Specificity: 100%*

CMC, Vellore, India: Sensitivity: 100%* | Specificity: 100%*

*This information is provided for the Scientific Community Enquiring for an independent evaluation other than company's in house evaluation.

It is not for commercial or promotional purpose.



Since 1969

J. Mitra & Co. Pvt. Ltd.

.....a vision to serve mankind[™]

A 180-181, Okhla Indl. Area, Phase-1, New Delhi - 110 020, INDIA | Tel: +91-11-47130300, 47130500
E-mail: jmitra@jmitra.co.in | Website : www.jmitra.co.in



ISO 13485:2016
ICMED 13485