

VITAMIN B12 iClia

Chemiluminescence Immunoassay for the Quantitative Detection of vitamin B12 (cyanocobalamin) in Human Serum

PROBLEM	POSSIBLE CAUSE	SOLUTION
2. High Vitamin-B12 test results	a) Use of turbid, lipaemic or hemolyzed sample.	Use clear fresh sample. Refer test specimen collection, handling and processing for more details.
	b) Sample position is wrongly defined while loading the sample details in analyzer.	check the sample position and run the test meticulously.
	c) Magnetic microsphere are not properly mixed before loading in the analyzer.	Ensure proper mixing of bottle containing microparticles by gentle shaking/ inversion before use.
	d) Wrong Sample identification	Make sample I.D. at the time of sample Collection.
3. Low Vitamin-B12 test results	a) Sample deterioration due to improper Storage or microbially contaminated sample.	Use clear fresh sample immediately after collection. Refer Specimen collection, and handling processing for more details.
	b) Sample position is wrongly defined while loading the sample details in analyzer.	check the sample position and run the test meticulously.
	c) Magnetic microsphere are not properly mixed before loading in the analyzer.	Ensure proper mixing of bottle containing microparticles by gentle shaking/ inversion before use.
	d) Wrong sample identification.	Mark the sample I.D. at the time of sample collection.

1. INTRODUCTION

Vitamin B12, also called cyanocobalamin, is a water-soluble vitamin that has a key role in the normal functioning of the brain and nervous system, and the formation of red blood cells. It is involved in the metabolism of every cell of the human body, especially affecting DNA synthesis, fatty acid and amino acid metabolism. Vitamin B12 is essential for the manufacture of red blood cells; a deficiency leads to a characteristic kind of anemia.

Vitamin B12 deficiency is most commonly caused by low intakes, but can also result from malabsorption, certain intestinal disorders, low presence of binding proteins, and use of certain medications. Vitamin B12 is rare from plant sources, so vegetarians are most likely to suffer from vitamin B12 deficiency. Infants are at a higher risk of vitamin B12 deficiency if they were born to vegetarian mothers. The elderly who have diets with limited meat or animal products are vulnerable populations as well.

Vitamin B12 deficiency can potentially cause severe and irreversible damage, especially to the brain and nervous system. At levels only slightly lower than normal, a range of symptoms such as fatigue, depression, and poor memory may be experienced. Vitamin B12 deficiency can also cause symptoms of mania and psychosis.

2. INTENDED USE

VITAMIN B12 iClia is a chemiluminescent microparticle immunoassay designed for in vitro quantitative detection of vitamin B12 (cyanocobalamin) in human serum. This kit is only operational in conjunction with J. Mitra CLIA Analyzer.















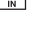

3. PRINCIPLE

VITAMIN B12 iClia is chemiluminescent immunoassay based on the "competitive" principle.

The Denaturation reagent, stabilizing reagent and samples are mixed, after incubation, neutralization reagent and Vitamin B12 antibody coated magnetic microparticles are added to reaction and incubate. After incubation, Vitamin B12 of a sample and Vitamin B12 of Acridinium ester completes for binding to anti vitamin B12 antibodies coated magnetic microparticle. After incubation and washing, Trigger and Pre-Trigger Solutions are added to the reaction mixture. The resulting chemiluminescent reaction is measured as relative Light units (RLUs). There is an indirect relationship between the amount of Vitamin B12 present in the sample and the RLUs detected by the optical system. Results are calculated automatically based on the established calibration curve.

4. DESCRIPTION OF SYMBOLS USED

The following are graphical symbols used in or found on J. Mitra diagnostic products and packing. These symbols are the most common ones appearing on medical devices and their packing. They are explained in more detail in European Standard EN ISO 15223-1:2021.

	Manufactured By		In vitro diagnostic medical device
	No. of tests		Instruction for use
	Lot Number Batch Number		Temperature Limitation
	Manufacturing Date		Caution, see instruction for use
	Expiry Date		Catalogue Number
	Keep away from sunlight		Do not use if package is damaged
	Contains biological Material of Human Origin		Contains biological Material of Animal Origin
	Country of Manufacture		Keep Dry

5. KIT PRESENTATION

- 50 Tests
- 100 Tests

6. KIT & ITS COMPONENTS

COMPONENT	DESCRIPTION
Reagent Tray-A	
Neutralization Reagent (RB)	Phosphate buffer solution with protein stabilizers.

Denaturation Reagent (RC)	Sodium hydroxide solution with protein stabilizers.
Stabilizing Reagent (RD)	Reducing agent solution with protein stabilizers.
Reagent Tray-B	
Magnetic (RA)	Magnetic microspheres coated with anti-VITAMIN B12 Microparticle Buffer antibodies with preservatives.
AE Conjugate (RD)	Vitamin B12 linked to acridinium ester with protein stabilizers.
Calibrator-1 (C0)	Low concentration of Vitamin B12 of in Human Serum containing preservatives.*RTU
Calibrator-2 (C1)	High concentration of Vitamin B12 of in Human Serum containing preservatives.*RTU
Control-1 (Q1)	Low concentration of Vitamin B12 of in Human Serum containing preservatives.*RTU
Control-2 (Q2)	High concentration of Vitamin B12 of in Human Serum containing preservatives.*RTU
Reagent Plugs	Silicon caps to cover the opened reagents.

7. STORAGE AND STABILITY

The shelf-life of the kit is 12 months from the date of manufacturing, when stored at 2-8°C. **Once the kit is opened, onboard stability of reagents, calibrator and control is 30 days at 2-8°C.**

8. ADDITIONAL MATERIAL AND INSTRUMENTS REQUIRED

- **Pre-Trigger Solution:** Hydrogen peroxide solution.
- **Trigger Solution:** Sodium hydroxide solution.
- **Wash Buffer:** Phosphate buffered saline solution with surfactant.
- **Assay Cup**
- **J. Mitra CLIA Analyzer**

All materials and analyzer to be used for running the Vitamin-B12 iClia shall be from J. Mitra & Co. Pvt. Ltd.

9. SPECIMEN COLLECTION & HANDLING

- Only human serum or plasma samples should be used for the test.
- For serum collection use serum vacutainer. While preparing serum samples, remove the serum from the clot as soon as possible to avoid hemolysis. Fresh serum/plasma samples are preferred.
- For plasma collection: use Dipotassium EDTA, Tripotassium EDTA, Sodium heparin and lithium heparin gel vacutainer.
- Specimens should be free of microbial contamination and may be stored at 2-8°C for one week, or frozen at -20°C or lower. Avoid repeated freezing and thawing.
- Do not use heat inactivated samples as their use may give false results. Hemolyzed and Icteric hyperlipemic samples may give erroneous results.
- Serum specimens from patients receiving anticoagulant or thrombolytic therapy may contain fibrin due to incomplete clot formation.
- Always use clear specimens. Centrifuge viscous/ thick or turbid specimen at 10,000 RPM for 15 minutes prior to use to avoid inconsistent result.
- Use of disposable pipettes or pipette tips is recommended to prevent cross contamination.

10. SPECIMEN PROCESSING

(A) FROZEN SAMPLE

VITAMIN B12 Clia test is best used with fresh samples that have not been frozen and thawed. However most frozen samples will perform well if the procedure suggested below is followed.

Allow the frozen sample to thaw in a vertical position in the rack. Do not shake the sample. This allows particles to settle to the bottom. Centrifuge the sample at 10,000 rpm for 15 minutes.

in vitro diagnostic Reagent, not for medicinal use

J. MITRA & CO. PVT. LTD.

A 180-181, Okhla Indl. Area, Phase-1, New Delhi-110 020, INDIA
Ph: +91-11-47130300, 47130500

e-mail: jmitra@jmitra.co.in Internet: www.jmitra.co.in

(B) TRANSPORTATION

If the specimen is to be transported, it should be packed in compliance with the current Government regulations regarding transport of aetiologic agents.

11. WARNING & PRECAUTION

CAUTION: THIS KIT CONTAINS MATERIALS OF HUMAN ORIGIN. NO TEST METHOD CAN OFFER COMPLETE ASSURANCE THAT HUMAN BLOOD PRODUCTS WILL NOT TRANSMIT INFECTION. NEGATIVE CONTROL, POSITIVE CONTROL & ALL THE SAMPLES TO BE TESTED SHOULD BE HANDLED AS THOUGH CAPABLE OF TRANSMITTING INFECTION.

- The use of disposable gloves and proper biohazardous clothing is STRONGLY RECOMMENDED while running the test.
- In case there is a cut or wound in hand, DO NOT PERFORM THE TEST.
- Do not smoke, drink or eat in areas where specimens or kit reagents are being handled.
- Tests are for in vitro diagnostic use only and should be run by competent person only.
- Do not pipette by mouth.
- All materials used in the assay and samples should be decontaminated in 5% sodium hypochlorite solution for 30-60 min. before disposal or by autoclaving at 121°C at 15psi for 60 minutes. Do not autoclave materials or solution containing sodium hypochlorite. They should be disposed off in accordance with established safety procedures.
- Wash hands thoroughly with soap or any suitable detergent, after the use of the kit. Consult a physician immediately in case of accident or contact with eyes, in the event that contaminated material are ingested or come in contact with skin puncture or wounds.
- Spills should be decontaminated promptly with Sodium Hypochlorite or any other suitable disinfectant.

12. PRECAUTIONS FOR USE & REAGENT HANDLING

- Do not use kit components beyond the expiration date which is printed on the kit.
- Store the reagents & samples at 2-8°C.
- Do not pool reagents from within a batch or between different batches, as they are optimised for individual batch to give best results.
- Before loading the reagent kit in the clia analyzer for the first time, ensure proper mixing of microparticle bottle to resuspend microspheres that may have settled during transport or storage.
- Once reagents are opened, reagent plug must be used to prevent reagent evaporation and contamination and to ensure reagent integrity. Reliability of assay results cannot be guaranteed if reagent plugs are not used according to the instructions given.
- Mark the test specimen with patient's name or identification number. Improper identification may lead to wrong result reporting.
- To avoid contamination, wear clean gloves when placing a reagent plug on an uncapped reagent bottle.
- Once a reagent plug has been placed on an open reagent bottle, do not invert the bottle as this will result in reagent leakage and may compromise assay results.
- Reagents may be stored on or off the Chemiluminescence immunoassay analyzer. If reagents are removed from the analyzer, store them at 2-8°C (with Reagent Plugs) in an upright position. For reagents stored off the system, it is recommended that they should be stored in their original trays and boxes to ensure they remain upright. If the microparticle bottle does not remain upright (with a Reagent Plug placed) while in refrigerated storage off the system, the reagent kit must be discarded.
- Run control-1 & control-2 in each assay to evaluate validity of the kit.
- Distilled or deionised water must be used for wash buffer preparation.
- Avoid strong light exposure during the assay.
- In case of any doubt the run should be repeated.

13. TEST PROCEDURE

Assay Procedure

- Refer to the Clia Analyzer user manual for detailed information on preparing the analyzer.
- Before loading the VITAMIN B12 iClia reagent kit on the analyzer for the first time, mix contents of the microparticle bottle to resuspend microspheres that may have settled during transportation/ storage. Once the microspheres have been loaded, no further mixing is required.

Important Note: Swirl the microparticle (RA) bottle 30 times. Visually inspect the bottle to ensure microspheres are resuspended. If microspheres are still adhered to the bottle, continue to Swirl the bottle until the microspheres have been completely resuspended. If the microspheres do not resuspend, DO NOT USE. Once the microspheres have been resuspended, remove the cap and place the reagent plug on the bottle to make it ready to use. Remove the cap of (RA), (RB), (RC) and (RD) bottles and place the reagent plugs before use as follow:

- | | | |
|-------------|---|--------------------|
| (RA) & (RB) | : | Natural color plug |
| (RC) | : | Purple color plug |
| (RD) | : | Brown color plug |
- Load the VITAMIN B12 iClia reagent kit on the Chemiluminescence immunoassay analyzer.
 - Verify that all necessary reagents are available in the reagent tray.
 - Ensure that adequate sample volume (not less than 250 µL) is present in sample tube prior to running the test.
 - Sample volume required for each additional test from same sample tube is 100 µL.
 - Ensure sample positions are properly defined at the time of loading in the analyzer.
 - The VITAMIN B12 test-specific parameters are stored in barcode placed on the reagent tray and read through barcode reader. In cases, the barcode cannot be read, contact customer support at: 011-47130300, 500 or write us at: jmitra@jmitra.co.in.
 - Mix VITAMIN B12 iClia calibrators and controls by gentle inversion before use. Open the cap and place the calibrators and control-1 & control-2 vials into each respective sample positions. Read the barcode for calibrator and controls provided with the kit.
 - Run calibration as mentioned in heading calibration below.
 - Press Run. The test result for first sample will be obtained at 37 minutes.
 - The Chemiluminescence immunoassay analyzer performs all the functions automatically and calculates the results.

Sample Dilution Procedures

- Samples with a Vitamin B12 conc. exceeding 2000 pg/mL shall be diluted 1:2 using the manual dilution procedure.
- Add 150 µl of the test sample to 150 µl of Sample Diluent iClia.
- The operator must enter the dilution factor in the patient or control order screen. The system will use this dilution factor to automatically calculate the concentration of the sample before dilution and report the result.

Calibration

- Traceability:** This assay has been standardized against the standard; NRCCRM, NIM-RM3603).
- Every Vitamin B12 iClia kit has a two-dimension code label containing the predefined master curve of the particular reagent lot.
- Test both the Calibrators in triplicate. Both control-1 (Q1) and control-2 (Q2) must be tested in each run to evaluate the assay calibration. Ensure that controls values are within the validity range specified in the Vitamin B12 iClia QC Bar code.
- Once calibration is accepted (within range) and stored, all subsequent samples may be tested without further calibration unless:
- Recalibrate the analyzer in following conditions:
 - After each exchange/use of new lot (Test reagent and Pre-trigger/ Trigger solution/wash buffer).
 - Every week and/or at the time of any component to be changed.
 - Controls are out of validation range.
 - Required by pertinent regulations.
 - After specified service procedures have been performed or maintenance to critical part or subsystems that might influence the performance of the VITAMIN B12 iClia.
- Calibration Range of VITAMIN B12 iClia kit is: 60 pg/ml to 2000 pg/ml.

RESULT CALCULATION:

The analyzer automatically calculates the concentrations of each sample either in pg/mL or pmol/L (selectable).

- Conversion factors:
- pg/mL × 0.738 = pmol/L
- pmol/L × 1.355 = pg/mL

14. EXPECTED VALUES

Each laboratory should establish its own range of normal value. The values given below are only indicative.

Population	Vitamin B12 (pg/mL)	Vitamin B12 (pmol/L)
New Born	160-1300	118-959
Adult	200-835	148-616
Adult > 60 years	110-800	81-590

15. PERFORMANCE CHARACTERISTICS

- Assay results obtained in individual laboratories may vary from data presented in this product insert.

Limit of Blank (LoB)

- The Limit of Blank was determined in accordance with the CLSI (Clinical and Laboratory Standards Institute) EP17-A requirements.
- The Limit of Blank is the 95th percentile value from n >20 measurements of analyte free samples over several independent series. The Limit of Blank corresponds to the concentration below which analyte-free samples are found with a probability of 95%.
- The observed LoB value was <50 pg/ml.

Accuracy: The accuracy of VITAMIN B12 iClia was detected with 60 clinical specimen and compared with Roche CLIA. The co-relation co-efficient is >0.970.

Precision

Intra Assay Variation

Within run variation was determined by 10 replicate measurements of two different VITAMIN B12 control sera (Low) and (High) in one assay in 3 different lots. The within assay variability is <5.5%.

Inter Assay Variation

Between run variation was determined by 10 replicate measurements in 10 sequential days of two different control sera (Low) and (High) in 3 different lots. The between assay variability is <10.0%.

Intra-Assay, n=10			Inter-Assay, n=10×3		
Control	Mean (pg/ml.)	CV	Sample	Mean (pg/ml.)	CV
1	260	5.35%	1	262.4	6.19%
2	514	7.10%	2	515.1	6.39%

Inter machine (CLIA Analyzer) Variation

Inter machine variation was determined by running 3 different lots with two different controls and 3 different CLIA Analyzer at a time. The inter machine variability is <15%.

Linearity

The linearity was determined in accordance with the CLSI (Clinical and Laboratory Standards Institute) EP6-A requirements.

The linearity range was verified by more than 6 concentration levels which encompass or be equal to the minimum and the maximum values of linearity range and duplicate assays in triplicate in single run for each lot at all 6 levels.

The VITAMIN B12 iClia kit has been demonstrated to be linear from 60 pg/ml. to 2000 pg/ml., regression (R2) of more than >0.990.

Specificity

Cross-Reactivity

A study was performed based on guidance from CLSI EP7-A2.

Potentially interfering substances were evaluated to determine whether VITAMIN B12 concentrations were affected when using the VITAMIN B12 iCLIA (Vitamin B12 Hormone assay) kit. Samples containing the potential interferents were prepared at two VITAMIN B12 concentrations. The samples were assayed, and the VITAMIN B12 concentrations of the spiked samples were compared to the reference samples.

Potential Interferent	Interferent Concentration	% Interferent Bias
Bilirubin	20 mg/dL	<10%
Hb	500 mg/dL	<10%
Intralipid	1000 mg/dL	<10%
Total protein	10 g/dL	<10%
RF	1000IU/mL	<10%
ANA	400AU/mL	<10%
HAMA	600ng/mL	<10%

16. LIMITATION OF THE TEST

- Results should be used in conjunction with other data; e.g., symptoms, results of other tests, and clinical impressions.
- If the VITAMIN B12 results are inconsistent with clinical evidence, additional testing is recommended.
- Specimens from patients who have received preparations of mouse monoclonal antibodies for diagnosis or therapy may contain human anti-mouse antibodies (HAMA). Such specimens may show either falsely elevated or depressed values when tested with assay kits that employ mouse monoclonal antibodies. Additional information may be required for diagnosis.
- Heterophilic antibodies in human serum can react with reagent immunoglobulins, interfering with in vitro immunoassays. Patients routinely exposed to animals or to animal serum products can be prone to this interference, and anomalous values may be observed. Additional information may be required for diagnosis.
- Rheumatoid factor (RF) in human serum can react with reagent immunoglobulins, interfering with in vitro immunoassays. Additional information may be required for diagnosis.
- The VITAMIN B12 iClia assay kit is susceptible to interference effects from triglycerides at > 500 mg/dL.

17. LIMITED EXPRESSED WARRANTY DISCLAIMER

The manufacturer limits the warranty to the test kit, as much as that the test kit will function as an in-vitro diagnostic assay within the limitations and specifications as described in the product instruction for use, when used strictly in accordance with the instructions contained therein. The manufacturer disclaims any warranty expressed or implied including such expressed or implied warranty with respect to merchantability, fitness for use or implied utility for any purpose. The manufacturer's liability is limited to either replacement of the product or refund of the purchase price of the product and in no case liable to for claim of any kind for an amount greater than the purchase price of the goods in respect of which damages are likely to be claimed.

The manufacturer shall not be liable to the purchaser or third parties for any injury, damage or economic loss, howsoever caused by the product in the use or in the application thereof.

18. REFERENCES

- J Clin Invest. 1972 Aug; 51(8): 1939–1949. Direct immunoassay of triiodothyronine in human serum.
- The Journal of Clinical Endocrinology & Metabolism, Volume 36, Issue 2, 1 February 1973, Pages 397–400, SERUM AND THYROID GLAND TRIIODOTHYRONINE IN THE HUMAN FOETUS.
- Iranian Journal of Reproductive Medicine Vol.10. No.1. pp: 47-52, January 2012 Comparison of serum levels of Tri-iodothyronine (T3), Thyroxine (T4), and Thyroid-Stimulating Hormone (VITAMIN B12) in preeclampsia and normal pregnancy.
- Archives of Disease in Childhood, 1975, 50, 565. Serum tri-iodothyronine, thyroxine, and thyrotrophin concentrations in newborns during the first 2 days of life.
- National Journal of Medical Research, Vol. 1, Issue 2 Oct – Dec 2011, Original article variation in thyroid hormones level among people of different age, gender and seasons, Piparia, Gujarat..

19. TROUBLE SHOOTING CHART

PROBLEM	POSSIBLE CAUSE	SOLUTION
1. Controls out of validation limit	a) Controls/ Calibrator deterioration due to improper storage or used after expiry. b) Cross contamination of Controls c) Reagents deterioration to improper storage or used after expiry.	Ensure calibration is done after 15 days and use controls/ Calibrator within 30 days once opened and check storage temp. It should be 2-8°C. Pipette carefully and do not interchange caps. Use reagents within 30 days once opened and Check storage temp. It should be 2-8°C.
	d) Magnetic microsphere are not properly mixed before loading in the analyzer.	Ensure proper mixing of bottle containing microparticles by gentle shaking/ inversion before use.